EasySampler[®]

Complete Stool Collection Kit

Easy and hygienic specimen collection

 Instructions for use

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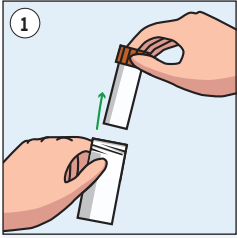
www.EasySampler.com


CE EasySampler[®] patented  GP Medical Devices A/S
Nupark 51, DK-7500 Holstebro • www.gpmd.us • Country of Origin: Denmark (EU)

For in vitro diagnostic use

EN**Step 1: Preparation and mounting of the EasySampler**

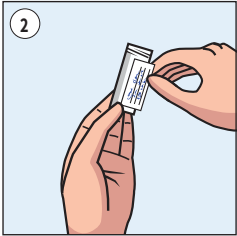Open the outer container and remove inner sample tube. Leave absorbent material in the outer container.


Fill out label with name, date of birth, date and time of sample collection.
Note: only add stool from one bowel movement per sample tube. Apply label to sample tube.

Put on disposable gloves.
Note: Gloves are not for patient examination or surgical use.

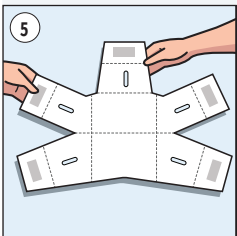Lift toilet seat and wipe off the surface of the toilet bowl rim.

Unfold the EasySampler containing five flaps. Orient the larger (REAR) flap towards the back of the toilet with the tape side down.
Note: Before affixing the EasySampler to the toilet bowl, bend the paper along the lines for better flexibility.

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

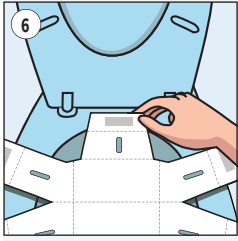
EN

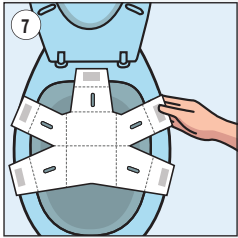
Remove the protective covering from the tape and adhere the REAR flap of the EasySampler onto the back of the toilet rim.

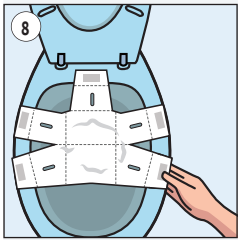Remove the protective covering from the two MIDDLE flaps and adhere them to the outside of the toilet bowl as far back as possible.
Note: EasySampler should not touch the surface of the water.

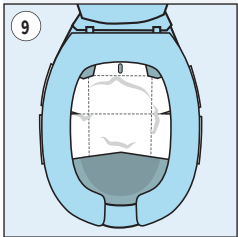Remove the protective covering from the two FRONT flaps and adhere them to the outside of the toilet bowl.

Make sure the EasySampler is taut and in the shape of a bowl. If not, re-orient the flaps for the correct positioning. Lower the toilet seat once the EasySampler is oriented correctly.
Note: Tape can be removed from toilet and re-adhered.

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EN

Have a bowel movement.
Note: Make sure stool does not come in contact with the toilet water or urine. If necessary, tilt EasySampler and slowly drain urine away from bowel movement before collecting a sample.

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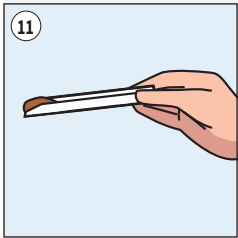
Step 2: Collecting the stool sample


Collect stool sample using one of the paper spoons.

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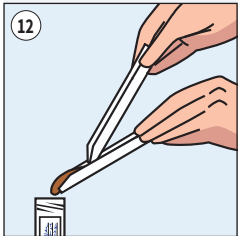
Use both paper spoons to push the sample into the SAMPLE tube.
Note: Be careful not to contaminate outside surface of sample tube.

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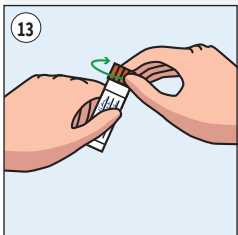
Screw the cap onto the sample tube. A plastic spoon is attached to the brown screw cap for removal of sample at time of testing.

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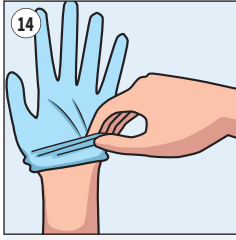EN


Hold used spoons in the palm of one hand. Remove glove over the spoons so the used material is contained inside the glove. Remove glove from other hand and properly dispose of material. Wash hands to prevent contamination.

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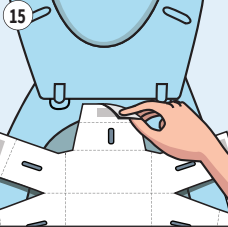Remove EasySampler flaps from toilet bowl and rim. Flush both the EasySampler and excess stool down the toilet.
Note: When using water saving toilets use a full flush. Repeat if necessary.

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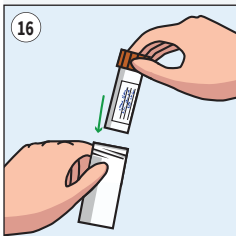
Step 3: Packing and sending the stool sample


Place SAMPLE tube into the OUTER container keeping the absorbent material between tube and container. Screw the lid onto the outer container.

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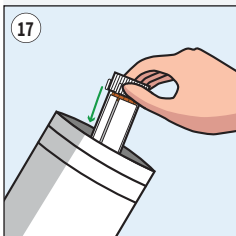
Place sealed container into padded envelope. Mail or carry sample package to laboratory as soon as possible.

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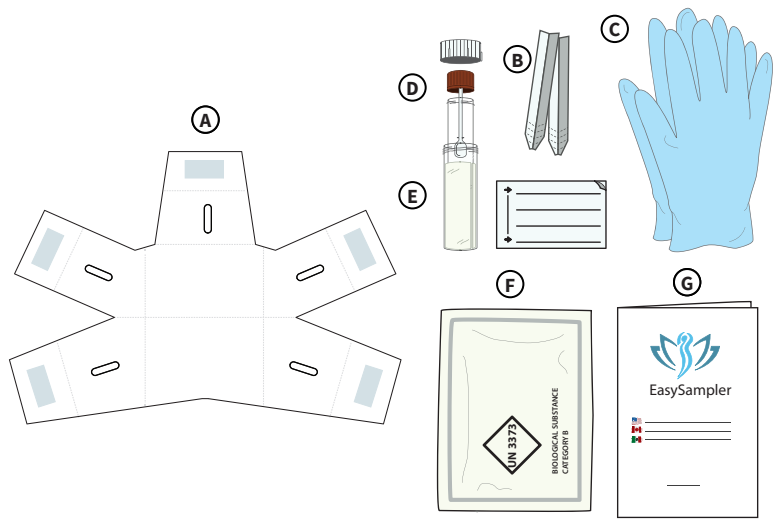EasySampler is available in the following packages:

Item no.	Description	
12 01 00	EasySampler – Single 1 x EasySampler 1 x Instructions for use	A G
12 60 10	EasySampler – Mini Kit - Single 1 x EasySampler 2 x EasySpoon 2 x Gloves – Nitrile – Size Large 1 x Instructions for use	A B C G
12 70 00	EasySampler – Complete Stool Collection Kit 1 x EasySampler 2 x EasySpoon 2 x Gloves – Nitrile – Size Large 1 x Sample tube 1 x Shipment container and label 1 x Envelope <UN3373> 1 x Instructions for use	A B C D E F G